CeraPlus™ Ring

- en Ostomy Barrier Rings
- de Stoma-Hautschutzringe
- es Anillos de barrera para ostomía
- fr Anneaux protecteurs pour stomie
- fr-CA Anneaux de champ protecteur de stomie
- it Anelli barriera per stomia
- nl Pastaringen voor stoma

EC REP
Hollister ULC
Foxford Road, Ballina
Co. Mayo, F26 F3X5, Ireland

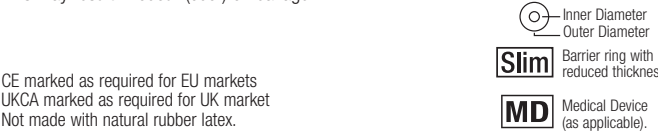
Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA
www.hollister.com/contact

Hollister
©2025 Hollister Incorporated.
12000285-825


en INSTRUCTIONS FOR USE

Contents: Ostomy barrier rings
Product Description: Ostomy barrier rings designed to improve the fit of an ostomy pouching system. Made with hydrocolloid skin adhesive. Infused with ceramide.
Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to help prevent drainage from getting under the skin barrier. Barrier rings help ostomy products conform to the skin.
WARNINGS: Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.


Notes: The product may be cut, stretched, or stacked together to help improve fit of a pouching system. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact Authorised (Authorized) Representative/ Responsible Person or local distributor.

To prepare skin and product

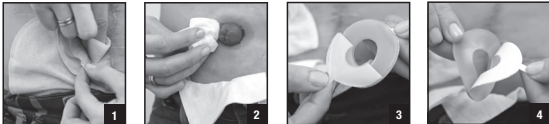To apply

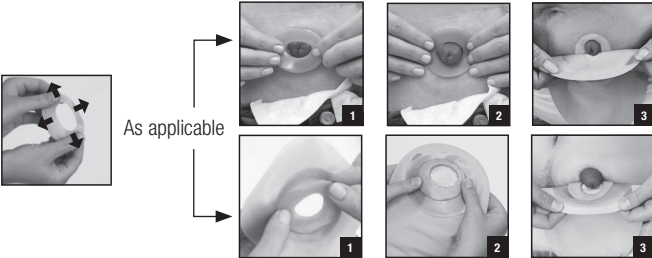To remove and dispose (Please follow all local disposal regulations)

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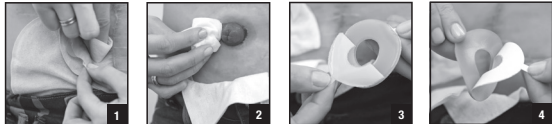
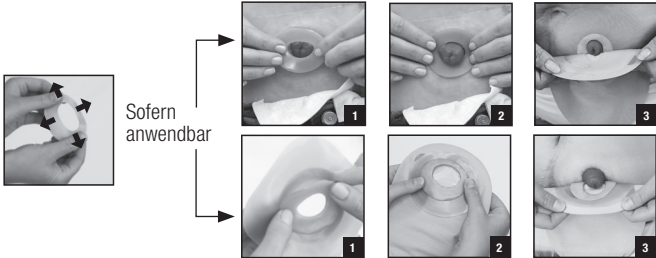
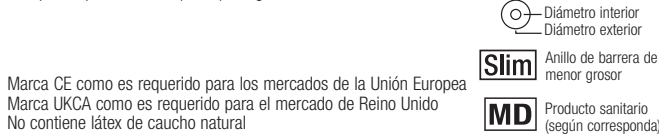
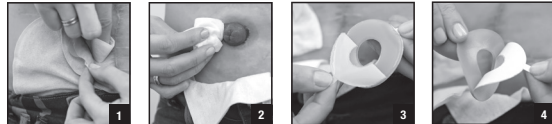
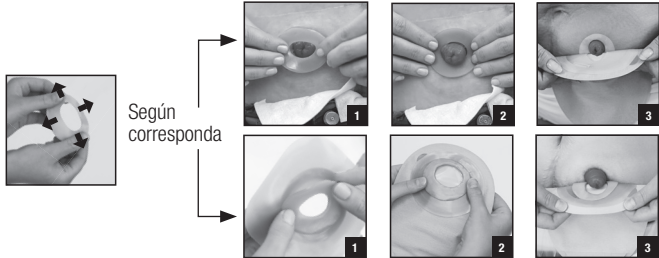
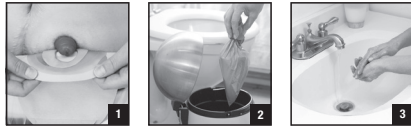
For reference only. Unauthorized use, disclosure or copying is prohibited.

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