US GB Instructions for Use

WARNING: To help reduce the potential for infection and/or other complications, do not reuse.

WARNING: If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

- Ready-to-use, single-use closed system intermittent catheter
- Hydrophilic-coated, Phthalate-free catheter
- Protective tip and sleeve supports no-touch insertion technique
- 2 smooth catheter eyelets
- Easy-to-remove ring cap
- Adhesive tap for adhering packaging to a surface
- 1000 mL collection bag
- Not made with natural rubber latex

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric (paediatric) patients who need to drain urine from the bladder.

PRECAUTION: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- Faux passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

PRECAUTION: Self-catheterization (catheterisation) should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization (catheterisation) frequency varies by person, the recommended frequency of your catheterization (catheterisation) should be provided by your healthcare practitioner. For any other questions about your catheterization (catheterisation), please contact your healthcare practitioner.

Instructions for Use

Wash your hands.

- To open package, pull top finger hole toward you and at least half way down. Use adhesive strip to tear package off if necessary to affix to a surface while preparing to catheterize (catheterise).
- MALE:** Hold the penis. Pull back foreskin (if present) and cleanse glans and opening of the urethra with mild, unscented soap or a non-alcoholic wet wipe. Remove catheter from package.
- Release collection bag by slipping paper band toward the catheter and off the bag or, **2b.**, with collection bag facing up, hold the catheter in one hand while pushing the bag away from the catheter with the other hand to break the paper band.
- Remove ring cap from protective tip of catheter.
- Hold the catheter in one hand, and with the other hand advance the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.
- Grasp catheter below protective tip plate, and insert protective tip until the plate comes in contact with urethral opening.

- Relax the foreskin (if present), **keeping protective tip in place.** Insert the catheter forward until the urine starts to flow. Once urine flow has stopped, slowly withdraw the catheter to ensure bladder is completely empty. Then gently remove catheter completely.

After catheterizing (catheterising), carry collection bag by handle.

- To empty the collection bag before disposing, tear at the arrow (Tear here to empty) to drain urine. Catheter and collection bag may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. Wash your hands.

- FEMALE:** Spread the labia and cleanse around opening of urethra with mild, unscented soap or a non-alcoholic wet wipe. Remove catheter from package.

- Release collection bag by slipping paper band toward the catheter and off the bag or, **3b.**, with collection bag facing up, hold the catheter in one hand while pushing the bag away from the catheter with the other hand to break the paper band.

- Remove ring cap from protective tip of catheter.
- Hold the catheter in one hand, and with the other hand advance the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.

- Part the labia with two fingers. Grasp catheter below protective tip plate, and insert protective tip until the plate comes in contact with urethral opening.

- Release the labia, **keeping protective tip in place.** Insert the catheter forward until the urine starts to flow. Once urine flow has stopped, slowly withdraw the catheter to ensure bladder is completely empty. Then gently remove catheter completely.

After catheterizing (catheterising), carry collection bag by handle.

- To empty the collection bag before disposing, tear at the arrow (Tear here to empty) to drain urine. Catheter and collection bag may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. Wash your hands.

NOTE: Store boxes in a flat position and at normal room temperature. This product consists of: polyurethane sleeve, thermoplastic elastomer tip, and tubing not made with DEHP.

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