Pre-Iubricated Urinary Catheterisation System	GB
Einmalkatheter im Auffangbeutel zur intermittierenden Selbstkatheterisierung	DE
Sistema de sondaje urinario prelubricado	ES
Set de sondage urinaire prélubrifiée	FR
מערכת צנרת עם מומה סגור	IL
Sistema chiuso per l'autocaterizzazione intermittente	IT
Sistema de Caterismo Urinario	PT
Pre-Iubricado	SA
كندیلندن یاجلی uriner kateterizasyon sistemi	TR

InstantCath Protect™

**STERILE**

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GB Instructions for Use

WARNING: To help reduce the potential for infection and/or other complications, do not reuse.

WARNING: If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

PRECAUTION: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False Passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

PRECAUTION: Self-catheterisation should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation frequency varies by person, the recommended frequency of your catheterisation should be provided by your healthcare practitioner. For any other questions about your catheterisation, please contact your healthcare practitioner.

- Ready-to-use, single-use closed system intermittent catheter
- Protective tip and collection bag supports no-touch insertion technique
- 2 smooth catheter eyelets
- Protection cap **(A)**
- Protective Tip **(B)**
- Introducer **(C)**
- Introducer plate **(D)**
- Holding mechanism **(E)**
- Catheter **(F)**
- Urine collection bag **(G)**
- Catheter funnel **(H)**
- Perforation/tear for emptying **(I)**
- Not made with natural rubber latex

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and paediatric patients who need to drain urine from the bladder.

Instructions for Use

Wash your hands.

1. Fully fold back the packaging by peeling the non-transparent side of the package.
2. Remove the plastic cap from the protective tip. The cap contains extra gel which can be used should additional lubrication be required. Wipe the protective tip around the inside of the cap to coat with the additional gel.
3. For easier insertion of the catheter, move the tip of the catheter forward into the protective tip.

Grasp the holding mechanism with one hand and with the other hand, move the catheter forward until the tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.
4. Lay the catheter on the opened packaging. The protective tip should not be outside the opened package.

- 5a. **MALE:** Hold the penis. Pull back the foreskin and wash the glans and the urethral opening. Grip the catheter through the holding mechanism and insert the protective tip until the plate contacts the glans. Slightly press the plate onto the glans in order to ensure that the protective tip does not slip out of the urethra. Release the foreskin.

- 5b. **FEMALE:** Spread the labia and cleanse. Grip the catheter through the holding mechanism and insert the protective tip until the plate contacts the urethral opening. Edge A of the urine collection bag should be above the catheter tube to ensure good drainage. Slightly press the plate against the urethral opening to ensure the protective tip does not slip out of the urethra. Release the labia.

As you advance the catheter and the pouch begins to gather, if you want to pull back the pouch, grip the catheter firmly through the holding mechanism. This action will help prevent the catheter from slipping out of the urethra before gently pulling back on the pouch.

Continue to insert the catheter forward until the urine starts to flow. Once urine has stopped flowing, grip the catheter through the holding mechanism and slowly withdraw the catheter.

6. To empty the urine, hold the bag above the perforation and tear towards the bottom of the bag. Empty the contents into a toilet or suitable receptacle.

NOTE: Store boxes in a flat position and at normal room temperature.

This product consists of: polyvinylchloride (PVC) catheter and funnel, thermoplastic elastomer and silicone tip components, and lubricating water/glycerin-based gel, and polyethylene/EVA collection pouch.

For reference only. Unauthorized use, disclosure or copying is prohibited.

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©2018 Hollister Incorporated.