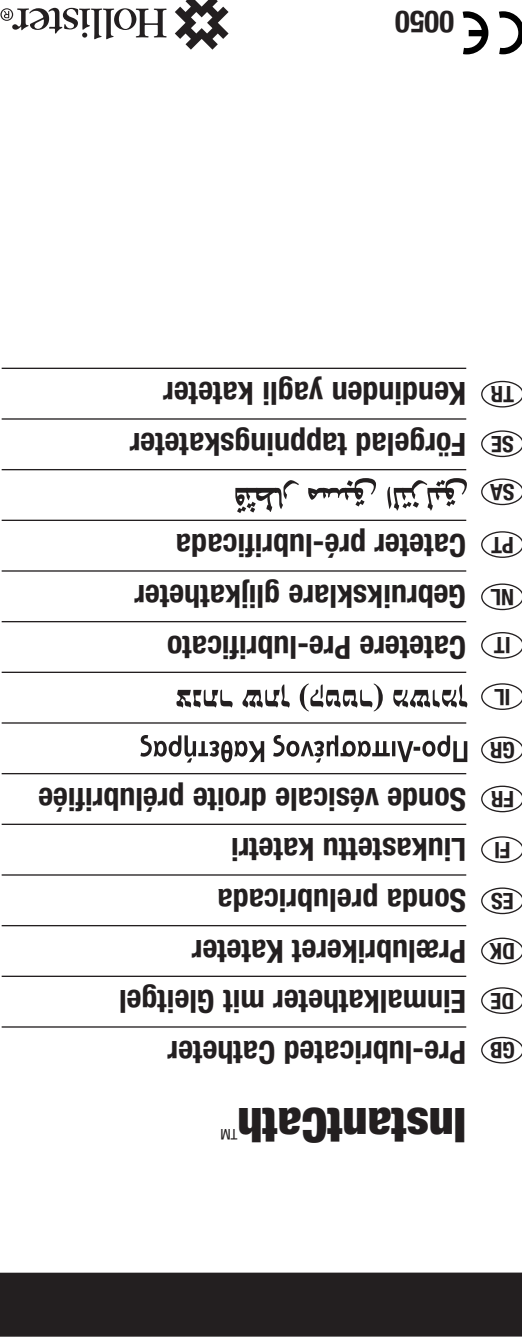
STERILE R

Sterilisation by irradiation
Sterilisation durch Bestrahlung
Steriliseret ved bestråling
Esterilización por irradiación
Sterilifitu säteilyttämällä
Sterilisation per irradiazione
Αποστείρωση με ακτινοβολία
עֲשֵׂיית בְּרַחֲנוּת הֶחֱרִיבָה
Sterilizazione con radiazioni
Sterilisatie door middel van bestraling
Esterilização por irradiação
Steriliserad genom bestrålning
İşnılama yoluyla sterilizasyon

Do not use if package is damaged
Nicht verwenden wenn Verpackung beschädigt
Må ikke anvendes, hvis pakningen er beskadiget
No utilizar si el envase está dañado
Ala káytá, jos pakkaus on vahingoittunut
Ne pas utiliser si l'emballage est endommagé
Να μην χρησιμοποιείται αν η συσκευασία έχει υποστεί ζημιά
Αλץ להשתמש אם האריזה פגומה
Non usare se l'involucro non è integro
Niet gebruiken indien de verpakking beschadigd is
Não utilizar se a embalagem estiver danificada
لا تستخدم العبوة إذا كانت معطوبة
Används ej om förpackningen är skadad
Paket hasar görmşu ise kullanmayın

Keep dry
Troocken lagem
Opbevaes tørt
Mantener en lugar seco
Säilytä kuivassa
A conserver à l'abri de l'humidité
Να φυλάσσεται σε ξηρό μέρος
לשמור במקום יבש
Conservare in luogo asciutto
Droog bewaren
Manter en lugar seco
تحتفظ جافة
Förvaras tørt
Kuru muhafaza edin

Number of units
Stück
Antal enheder
Número de unidades
Kappalemäärä
Nombre d'unités
Αριθμός τεμαχίων
ספר יחידות
Numero di unità
Aantal in verpakking
Número de unidades
عدد الوحدات
Antal enheter
Birim sayı

Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA
Hollister ULC
Foxford Road, Ballina
Co. Mayo, Ireland

Consult Instructions for Use
Siehe Gebrauchsanleitung
Se brugsvejledning
Consulte las instrucciones de uso
Katso käyttöohjeista ohjeet käyttöä varten
Consultar le mode d'emploi
Συμβουλευτείτε τις Οδηγίες Χρήσης
עֲיִן בְּרוּרֹאת הִישִׁימוּשׁ
Consultare le istruzioni d'uso
Leest u de gebruiksaanwijzing
Consulte as Instruções de Uso
اطلع على تعليمات الاستخدام
Bruksanvisning för sjukvårdspersonal
Kullanma Talimatına Bakın

Do not reuse
Nur zum einmaligen Gebrauch
Bar ikke genbruges
No reutilizar
Ala káytää uudelleen
Ne pas réutiliser
Να μην επαναχρησιμοποιείται
אף לעשות שימוש חוזר במוצר
Non riutilizzare
Niet hergebruiken
Não reutilizar
لا تـد استخدامه مرة أخرى
Abrangwini into
Tekrar kullanmayın

Keep away from heat
Vor Wärme schützen
Undgå direkte sol
Mantener fuera de la luz directa
Suojattava kuumuudelta
A conserver à l'abri de la chaleur
Να φυλάσσεται μακριά από θερμότητα
לשמור מרחוק מהחום
Tenere lontano da fonti di calore
Beschermen tegen warmte
Manter afastado de fontes de calor
تحتفظ بعيدا عن الحرارة
Skyddas mot värme
Isädan uzak tutun

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
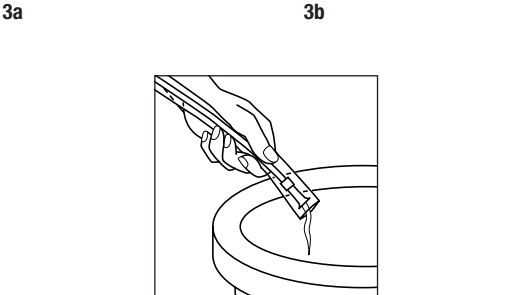
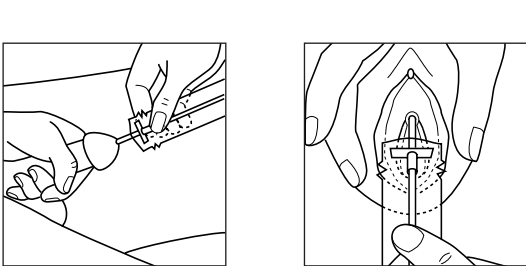
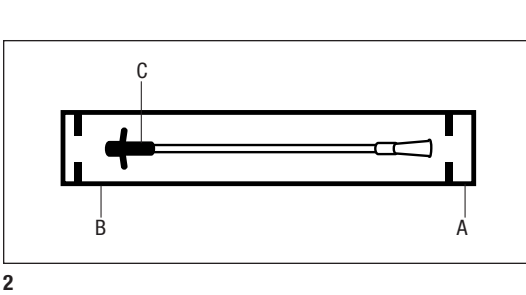
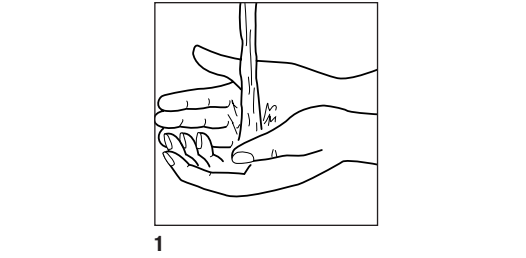
GB Instructions for Use

WARNING: To help reduce the potential for infection and/or other complications, do not reuse.

WARNING: If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

PRECAUTION: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False Passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

PRECAUTION: Self-catheterisation should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation frequency varies by person, the recommended frequency of your catheterisation should be provided by your healthcare practitioner. For any other questions about your catheterisation, please contact your healthcare practitioner.

- Ready-to-use, single-use intermittent catheter
- Protective tip and sleeve supports no-touch insertion technique
- 2 smooth catheter eyelets
- Not made with natural rubber latex

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and paediatric patients who need to drain urine from the bladder.

Instructions for Use

- Wash your hands.
- Tear open funnel end of package A. Tear open tip end at B. Clean the urethral meatus and surrounding area. Advance catheter tip approximately 5-6 cm beyond top end of package. Direct funnel end towards toilet or collection device.
- MALE:** Use one hand to hold and straighten penis and the other hand to grasp the catheter below holding mechanism C (Fig. 2). Introduce catheter into urethral meatus.
- FEMALE:** Use one hand to spread labia and the other hand to grasp the catheter below holding mechanism C (Fig. 2). Introduce catheter into urethral meatus.
- Move one hand to shaft of catheter approximately 5 cm behind holding mechanism C (Fig. 2) and begin advancing until urine starts to drain through package.

MALE: It will be helpful to occasionally straighten the package while advancing the catheter. This will prevent the catheter from backing out of the urethra.

When urine flow stops, ease catheter out and dispose of catheter and package.

NOTE: Store boxes in a flat position and at normal room temperature.

This product consists of: polyvinylchloride (PVC) catheter and funnel, polypropylene/thermoplastic elastomer guide mechanism, and water soluble lubricating gel.

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©2018 Hollister Incorporated.