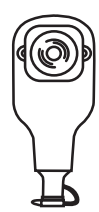
Flextend™

- en

1 Piece High Output Drainable Ostomy Pouch
- da

1-dels high output tømbar stomipose
- de

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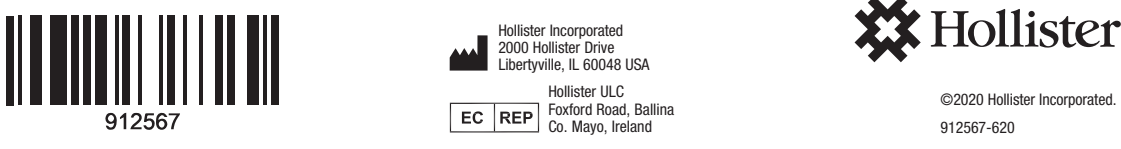
EN INSTRUCTIONS FOR USE

Contents

- High output drainable ostomy pouch with soft tap and attached Flextend skin barrier.
- Measuring guide for cut-to-fit products. Use for pre-sized products is optional.

Product Description: Drainable pouch with a durable skin barrier suitable for extended wear. The pouch is designed to be emptied. The pouch is most suitable for colostomies or ileostomies including high output stomas or post-operative use. May be connected to a Hollister high output collection bag.

Skin barriers are available in many styles:

- a flat skin barrier is best suited for a protruding stoma.
- a convex or soft convex (curved) skin barrier is best suited for a stoma that does not protrude. Convex pouches have belt tabs (belt not included).
- a cut-to-fit product can be cut by the user to fit the stoma.
- a pre-sized product is provided pre-cut for convenience.
- a tape (adhesive) border helps provide an extra sense of security around the edge of the skin barrier.

Made with hydrocolloid skin adhesive, polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use: This product is intended to collect output from a stoma (e.g., colostomy, ileostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage. Ensure proper fit as instructed. If the barrier opening is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

Notes: To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Frequent removal of a Flextend skin barrier in a short period of time (e.g., multiple times a day) may result in skin damage.

Product fit is affected by many factors such as stoma construction, stoma output, skin condition, and abdominal contours. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation.

In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

MD Medical Device (as applicable)

Not made with natural rubber latex.
CE mark as required

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