2 Piece Urostomy Pouch

2-dels urostomipose

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2-osainen virtsa-avannepussi

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INSTRUCTIONS FOR USE

Contents

- Urostomy pouch with attached flange.
 - Ostomy belt tabs (belt not included)
- Urostomy drain tube adapter for use to connect pouch to a drainage bag / leg bag (not included) if desired.
- Plastic caps to cover the tap for a sense of security from drips.
- Disposal bags

Product Description: Part of a two-piece ostomy pouching system. Designed to be used with a corresponding skin barrier with the same flange size. The pouch is designed to be emptied. The pouch is most suitable for urostomies. It has a tap to empty liquid. Product is made of polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use: This product is part of a two-piece ostomy pouching system intended to collect output from a stoma (e.g., urostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse by reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

Notes: Excess mucus from a urostomy stoma may cause the pouch to drain slower than expected. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

MD Medical Device
(as applicable).

Not made with natural rubber latex.
CE mark as required

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©2019 Hollister Incorporated.