- en** 2-Piece Drainable Pouch with soft tap
- da** 2-dels tømbar pose med blød udløbsstuds
- de** Zweiteiliger Ausstreifbeutel mit weichem Auslass
- es** Bolsa abierta de 2 piezas con tapón blando
- fi** 2-osainen tyhjennettävä avanpussi, jossa pehmeä hana
- fr** Poche vidable avec bouchon souple 2 pièces
- fr-ca** Sac vidable 2 pièces avec valve souple
- it** Sacca per ileostomia per sistema due pezzi con rubinetto soft
- nl** Tweedelig open opvangzakje met zacht afvoerkraantje
- no** Todelt tømbar pose med myk tømmestuss
- sv** 2-dels tömbar påse med mjuk tappningskran

www.hollister.com/symbolglossary

Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60040 USA
Hollister LLC
Foxford Road, Ballina
Co. Mayo, Ireland

INSTRUCTIONS FOR USE

Contents

- Drainable pouch with soft tap and attached flange
 - Ostomy belt tabs (belt not included)
- Clamp closure (see product label)
- Disposal bags

Product Description: Part of a two-piece ostomy pouching system. Designed to be used with a corresponding skin barrier with the same flange size. The pouch is designed to be emptied. The pouch is most suitable for colostomies or ileostomies including high output stomas or post-operative use. Product is made of polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use: This product is part of a two-piece ostomy pouching system intended to collect output from a stoma (e.g., colostomy, ileostomy) or skin opening (e.g., wound, fistula).

WARNINGS

Reuse by reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.



Notes: Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

MD Medical Device (as applicable).


Not made with natural rubber latex. CE mark as required


To attach pouch to barrier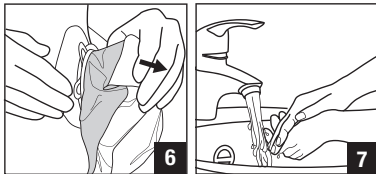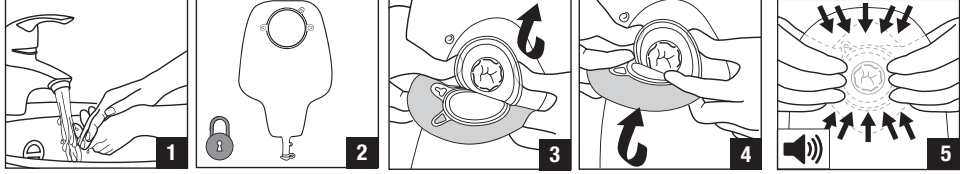

To empty


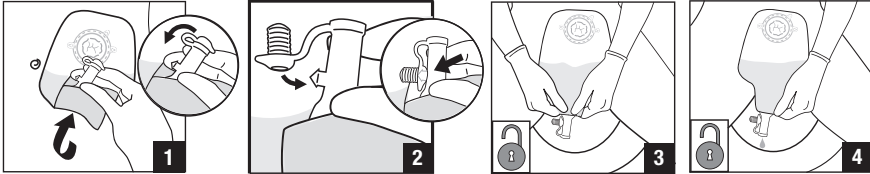
Pinch to control flow

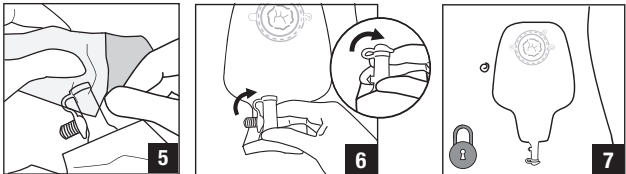To connect to a Hollister night drainage bag (not included) please refer to the instructions for use packed with that product

To use clamp closure (optional, if included with the product, see product label). If stoma output thickens, use a clamp closure.

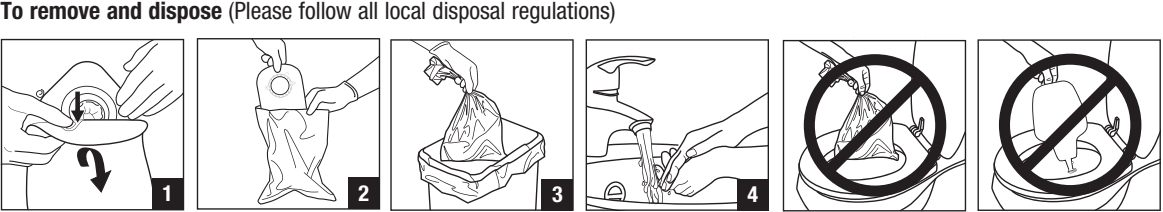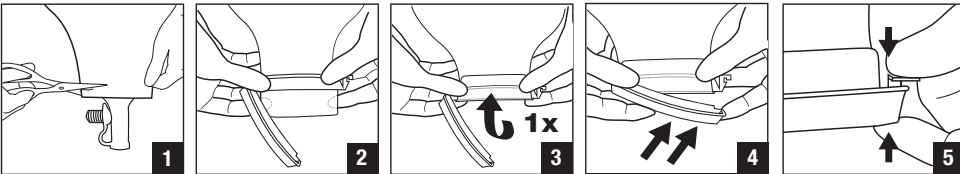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