en Medical Adhesive Remover


Contents: Medical adhesive remover

Product Description: Silicone-based medical adhesive remover spray. Alcohol free – no sting! Can be sprayed from any direction.

Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to ease removal of an ostomy skin barrier or adhesive residue.

INGREDIENTS: HEXAMETHYLDISILOXANE

WARNINGS

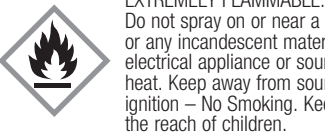EXTREMELY FLAMMABLE: Do not spray on or near a flame or any incandescent material, electrical appliance or source of heat. Keep away from sources of ignition – No Smoking. Keep out of the reach of children.

Precaution

Single patient – multiple use. Sharing the product with multiple users creates risk of infection.

Precautions: For external use only. Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs.

Pressurised (pressurized) container. Protect can from sunlight and do not expose to or store at temperatures exceeding 122°F (50°C). Do not puncture or incinerate container, even after use. DO NOT USE IN CONFINED SPACES.

Notes: Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation.

In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

INSTRUCTIONS FOR USE

- Spray remover around the edge of the adhesive.
- After a few seconds gently peel the adhesive away from the skin using additional medical adhesive remover if necessary.

Dries in seconds. Once dried, medical adhesive remover will not affect adhesion of products applied later.

Alternate application: saturate a gauze pad and wipe the adhesive away. Remover evaporates quickly, additional applications to gauze may be required.

May use until can is empty. Please follow all local disposal regulations.

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