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Contraindications

Male Incontinence Sheath (Catheter), Self-Adhesive

Product Description

Self-adhering latex male external sheath (catheter) suitable for extended wear. Made with latex and adhesive.

Indications and Intended Use

The product is an accessory intended to channel urine from the penis to extension tubing and urinary collection bags for adult males with urinary incontinence.

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion, leakage, and skin irritation.

Precautions

This product contains natural rubber latex which may cause allergic reactions. Do not use on irritated or compromised skin. Do not use if allergic reaction occurs.

Ensure proper fit as instructed. If the sheath (catheter) size is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time. Follow directions to remove if experiencing swelling, numbness, discomfort, pain, discoloration, or abnormal appearance.

Notes

For good hygiene, change sheath (catheter) daily.

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use lotions, powders, or ointments on the skin as these products may interfere with adhesion of the product. Contact a healthcare practitioner for help with the product or with skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

1. Verify correct product size prior to use. Trim pubic hair if necessary.
2. Wash penis with mild soap and warm water. Dry thoroughly.

- 1 Open package at perforation. The sheath (catheter) is supplied with a plastic collar, which presents the inner flap in a position for application.
- 2 Pinch sheath (catheter) closed with the thumb and forefinger in the finger-cuts.
- 3 Place on the penis, so the tip of the glans is against the inside of the sheath (catheter), which is pinched with thumb and forefinger. The glans should protrude into the opening of inner flap, which should fit snugly on the glans to prevent backflow of urine. Do not position inner flap too far forward. Do not position the entire glans through the inner flap. If applying to an uncircumcised penis, allow foreskin to remain in its natural position. Take care not to pinch or retract the foreskin.
- 4 Slowly unroll onto the penis with as little wrinkling as possible until product is fully extended. Do not force the plastic collar onto the penis. Remove and discard the plastic collar after sheath (catheter) is in place.
- 5 Seal excess material to itself by pinching it together, thus closing any leakage channels and providing a secure seal.
- 6 Gently squeeze to properly seal adhesive to the skin. Do not attempt to reposition the product.

To connect this product to a drainage system, refer to the instructions for use packed with that product.

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