en Urinary leg bag straps/holsters

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en INSTRUCTIONS FOR USE

Contents

- Urinary leg bag straps or holsters (see product label)

Product Description: Straps or holsters to hold a urinary leg bag securely to the thigh or calf.

Not made with natural rubber latex.

Holsters: Product is made of Polyester and elastane.

Straps: Product is made of Elastane, nylon, and silicone.

Indications and Intended Use: The straps or holsters secure a urinary leg bag to the leg. The straps or holsters are used by individuals with urinary incontinence or a urinary stoma.



Precautions

- Single patient – multiple use. Sharing this product with multiple users creates a risk of infection.
- To ensure adequate circulation, do not overtighten leg bag straps. Check the area under the straps or holsters regularly for skin irritation or trauma.
- To prevent improper drainage do not attach leg bag to knee.



Notes: Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Applying the leg bag straps

Adjust the straps so they are comfortable and secure.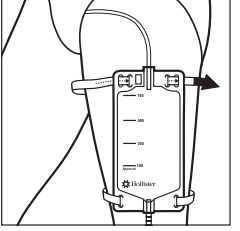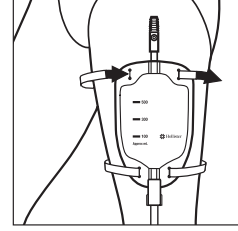

Applying the leg bag holsters

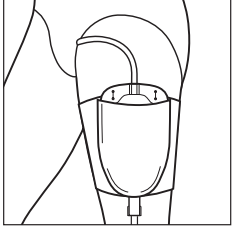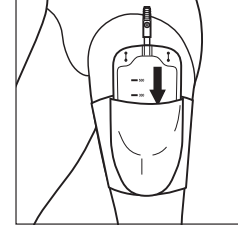To remove and dispose (Please follow all local disposal regulations)

Discard when visibly damaged or stained.

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©2020 Hollister Incorporated.