- en

Urinary leg bag holsters
- de

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Contents

- Urinary leg bag holsters

Product Description: Holsters to hold a urinary leg bag securely to the thigh or calf. Not made with natural rubber latex. Product is made of Nylon, elastane, cotton, polyester, and silicone.

Indications and Intended Use: The holsters secure a urinary leg bag to the leg. The holsters are used by individuals with urinary incontinence or a urinary stoma.

Precautions

- Single patient – multiple use. Sharing this product with multiple users creates a risk of infection.
- To ensure adequate circulation, do not overtighten.
 - Check the area under the holsters regularly for skin irritation or trauma.
 - To prevent improper drainage do not attach leg bag to knee.

Notes: Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

MD Medical Device (as applicable).

Not made with natural rubber latex.
CE mark as required

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

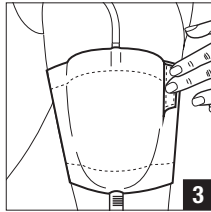
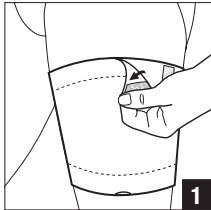
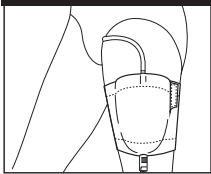
Upper Leg Holster

Lower Leg Holster


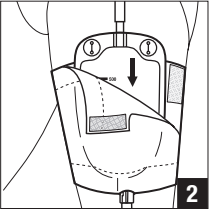
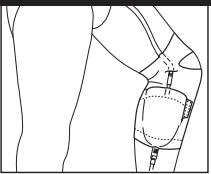
To remove and dispose (Please follow all local disposal regulations)
Discard when visibly damaged or stained.

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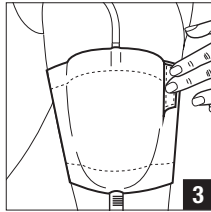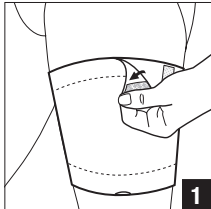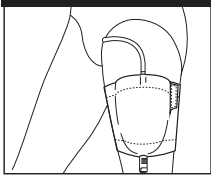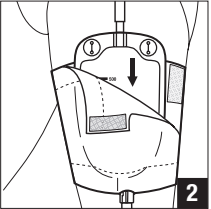
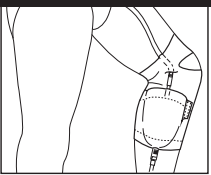
For reference only. Unauthorized use, disclosure or copying is prohibited.

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©2020 Hollister Incorporated.