- en

Deluxe urinary leg bag straps
- de

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Contents

- Deluxe urinary leg bag straps

Product Description: Straps to hold a urinary leg bag securely to the thigh or calf. Not made with natural rubber latex. Product is made of Nylon, elastane, polyester, elastic polyurethane fabric, and silicone.

Indications and Intended Use: The straps secure a urinary leg bag to the leg. The straps are used by individuals with urinary incontinence or a urinary stoma.

Precautions

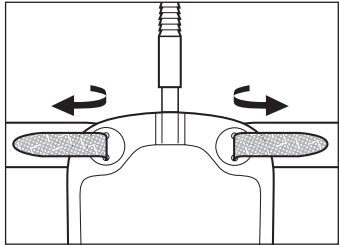
- Single patient – multiple use. Sharing this product with multiple users creates a risk of infection.

- To ensure adequate circulation, do not overtighten. Check the area under the straps regularly for skin irritation or trauma.
- To prevent improper drainage do not attach leg bag to knee.

Notes: Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Applying the leg bag straps

Adjust the straps so they are comfortable and secure.

To remove and dispose (Please follow all local disposal regulations)

Discard when visibly damaged or stained.

For best results, attach hook fastener to loop material before washing.

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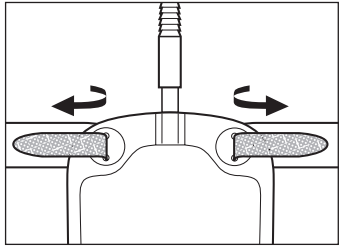
MD

Medical Device (as applicable).
CE mark as required

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Lufttrocknen

Nicht bügeln

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For reference only. Unauthorized use, disclosure or copying is prohibited.

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