- en

Urinary Collection Bags
- de

Urinauffangbeutel
- fr

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Urineopvangzakken

en

INSTRUCTIONS FOR USE

Contents

- Urinary collection bag with attached tubing

Product Description: Urinary collection bag that is non-drainable and should be disposed of after use. Can be held securely to a bed frame or stand using a hanger fixation (not included).
Not made with natural rubber latex. Product is made of polyvinylchloride (PVC), low-density polyethylene (LDPE), and polypropylene.

Indications and Intended Use: A urinary collection bag collects and contains urine for individuals with urinary incontinence or a urinary stoma.

WARNINGS To help reduce the potential for infection and/or other complications, do not reuse.

Notes: Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Applying the collection bag
Remove cap from stepped mount connector if present. Connect the stepped mount connector to the sheath (catheter). Make sure tubing is not kinked or obstructed. To attach to a night drainage bag hanger (not included) please refer to the instructions for use packed with that product. Please keep the bag off the floor.
To remove and dispose (Please follow all local disposal regulations)
Remove and discard after use.

MD

Medical Device (as applicable).
CE mark as required

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