en INSTRUCTIONS FOR USE

- Contents**
- Hydrophilic Urethral Intermittent Catheter
- Product Description:**
- Ready-to-use hydrophilic intermittent catheter with colour-coded funnel.
 - Gripper on 40cm catheter aids insertion and supports no-touch insertion technique.
 - Adhesive tab for adhering packaging to a surface.

	Catheter effective length: the length of the catheter that can be inserted into the urethra.	
40cm catheter	37 cm	370 mm
17cm catheter	12 cm	120 mm

Not made with natural rubber latex. Made with thermoplastic (gripper, 40cm catheter only), thermoplastic elastomer tubing and funnel.

Indications and Intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and paediatric patients who need to drain urine from the bladder.

WARNINGS

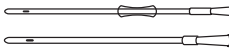
Do not use if package is damaged or opened. To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

Self-catheterisation should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation frequency varies by person, the recommended frequency of your catheterisation should be provided by your healthcare practitioner. For any other questions about your catheterisation, please contact your healthcare practitioner.

Notes: Store boxes in a flat position and at normal room temperature. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Urethral Intermittent Catheter

Single sterile barrier system

Consult Instructions for Use if package is damaged

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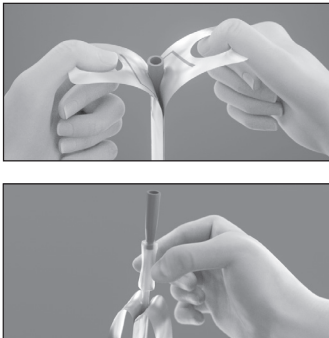
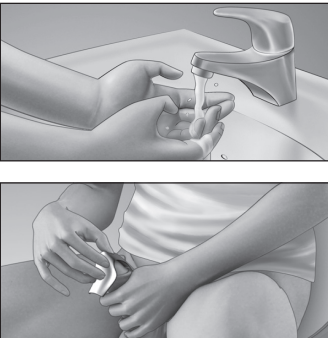
Stück

Medizinprodukt (sofern anwendbar)

For reference only. Unauthorized use, disclosure or copying is prohibited.

en Male Instructions	
To prepare Wash hands. Use mild, unscented soap or a non-alcoholic wet wipe to cleanse glans and opening of urethra as instructed by your healthcare practitioner. Choose a position to catheterise that is most comfortable for you (standing, sitting on the toilet, in wheelchair).	Using finger holes peel open the package until the funnel and gripper are visible. Hold funnel and/or gripper to remove catheter. Use adhesive strip on catheter package, if necessary, to affix to a surface while preparing to catheterise.

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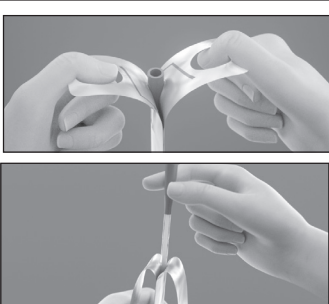
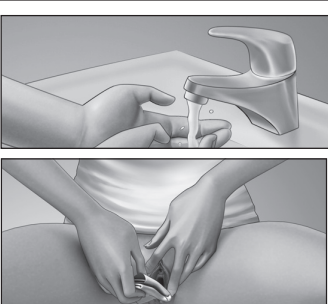
en Female Instructions	
To prepare Wash hands. Spread the labia and cleanse around opening of urethra with mild, unscented soap or a non-alcoholic wet wipe as instructed by your healthcare practitioner. Choose a position to catheterise that is most comfortable for you (standing, sitting on the toilet, in wheelchair or lying in bed).	Using finger holes peel open the package until the funnel is visible. Hold funnel to remove catheter. Use adhesive strip on catheter package, if necessary, to affix to a surface while preparing to catheterise.

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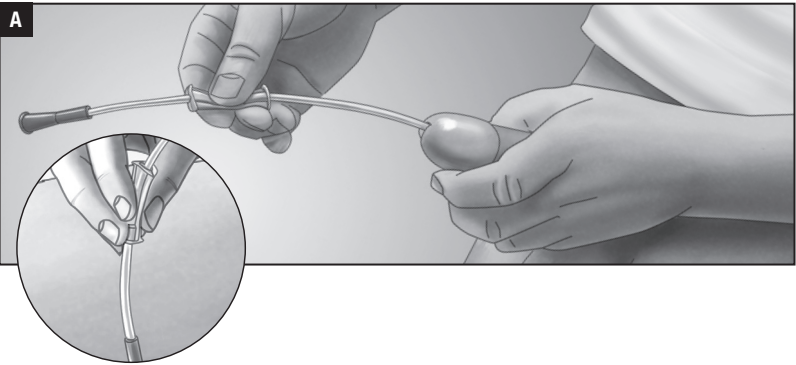
To use Pull back foreskin if present. A : Pull gripper away from the funnel. Hold gripper anywhere along the catheter, so you do not touch the catheter. With the penis stretched and in upright position, hold gripper tightly to begin inserting catheter. Slightly release gripper, and slide it back towards funnel. Then hold gripper tightly again to advance the catheter. Repeat until urine starts to flow. Relax foreskin, if present.	
--	--

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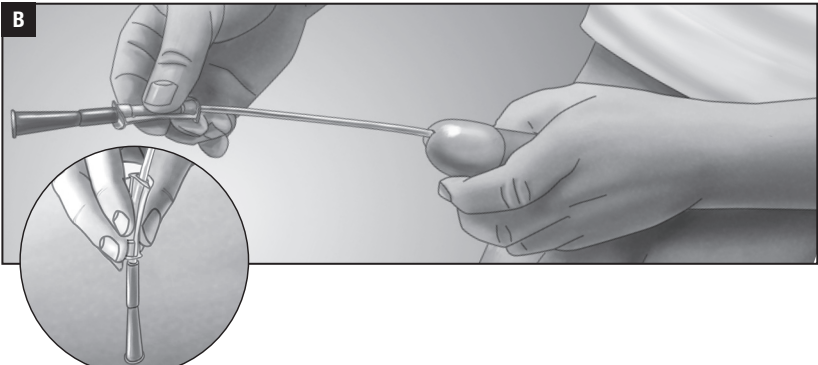
B : Alternative insertion option: keep the gripper 'docked' with the funnel. With penis stretched and in upright position, hold gripper/funnel and insert catheter into the urethra with one hand until urine starts to flow. Relax foreskin, if present.	
--	--

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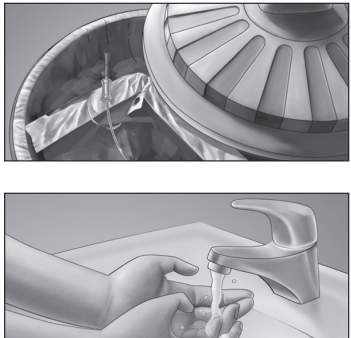
To remove and dispose (Please follow all local disposal regulations) Once urine flow has stopped slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. Catheter may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. Wash hands.	
---	--

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