en INSTRUCTIONS FOR USE

Contents
Hydrocolloid skin barrier extenders

Product Description
Barrier extenders are thin, flexible strips designed to help an ostomy pouching system conform to uneven skin folds and contours. They are designed to help reduce skin barrier edge lifting and provide an extra sense of security. Product is made with hydrocolloid skin adhesive suitable for general wear.

Indications and Intended Use
This product is an accessory to an ostomy pouching system intended to help an ostomy product conform to the skin.

WARNINGS
Reuse of a single use product may create potential risk to the user such as poor adhesion.

Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage.

Notes:
To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

MD

Medical Device
(as applicable)

Number of Units

Not made with natural rubber latex.
CE mark as required

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