
en INSTRUCTIONS FOR USE


To prepare
1. Assemble all equipment and prepare the area. Wash your hands as advised by your healthcare practitioner.
2. Choose a position to catheterise that is most comfortable for you (standing, sitting on the toilet, in wheelchair or lying in bed).


3. Spread the labia with two fingers of your non-dominant hand. Use mild, unscented soap and water or a non-alcoholic wet wipe to cleanse around opening of urethra as instructed by your healthcare practitioner.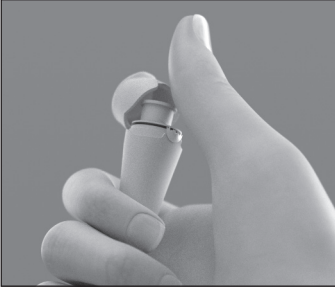


To use
4. Open the case by flipping up the cap, with your dominant hand, which breaks the seal. Leave the product in the case. NOTE: if seal is already broken, do not use the catheter. If you are using a collection bag accessory (not included), attach to the funnel end of the catheter. No need to remove catheter from the case.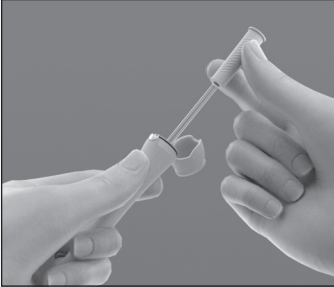

5. Hold the case with your non-dominant hand and remove catheter from the case with your dominant hand, touching only the funnel.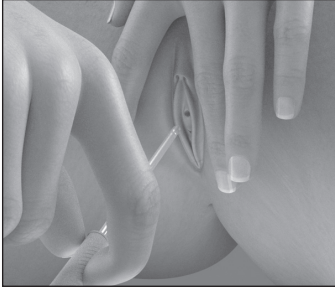


6. Part the labia with two fingers of your non-dominant hand. Touching only the funnel, insert the catheter with your dominant hand into the urethral opening, until urine starts to flow.

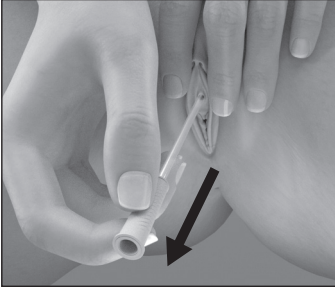7. Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. If desired, you may place the used catheter back inside the case and close the cap.

To dispose. Please follow all local disposal regulations.
8. You may dispose as appropriate in a recycling bin or waste bin immediately or later when it is more convenient. The recycling symbol on the Infyna Chic label applies only to the catheter case.

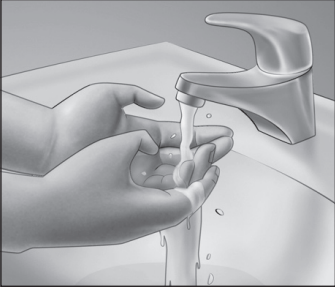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