VaPro Plus™

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CE 0050

en INSTRUCTIONS FOR USE

Contents
No touch urethral intermittent catheter with integrated collection bag
Product Description

- Ready-to-use hydrophilic intermittent catheter with colour (color)-coded funnel
- Protective tip and sleeve supports no-touch insertion technique.
- 1000 ml integrated collection bag
- Adhesive tab for adhering packaging to a surface.

Catheter effective length: the length of the catheter that can be inserted into the urethra.			
20 cm / 8" catheter	17 cm	170 mm	6.7"
40 cm / 16" catheter	37 cm	370 mm	14.6"

Not made with natural rubber latex. This product consists of: polyurethane sleeve, thermoplastic elastomer tip, and tubing not made with DEHP.

Indications and Intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and paediatric (pediatric) patients who need to drain urine from the bladder.

WARNINGS
Do not use if package is damaged or opened. To help reduce the potential for infection and other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

Self-catheterisation (catheterization) should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation (catheterization) frequency varies by person, the recommended frequency of your catheterisation (catheterization) should be provided by your healthcare practitioner. For any other questions about your catheterization (catheterization), please contact your healthcare practitioner.

NOTE: Store boxes in a flat position and at normal room temperature. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

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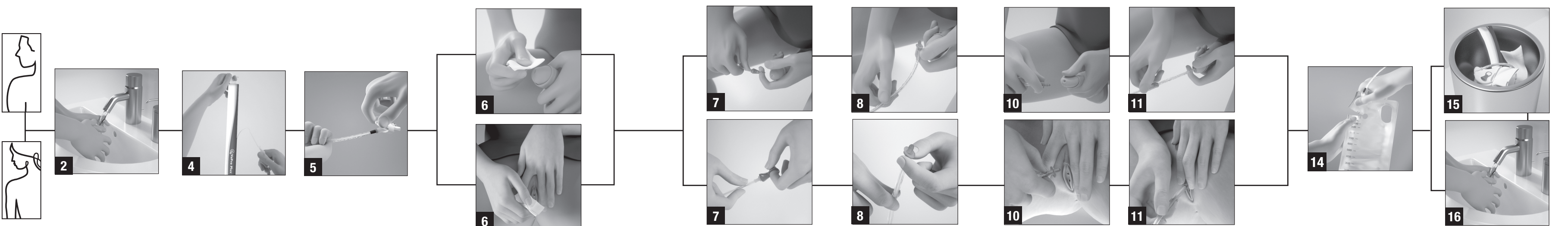
en Instructions To prepare

- Assemble all equipment and prepare the area. If helpful, use the sticky tab to adhere package to a surface.
- Wash your hands as advised by your healthcare practitioner.
- Position yourself to carry out intermittent catheterisation (catheterization). Manipulate your clothing to access your genitalia.
- Open the package. To open package, pull top finger hole toward you and at least halfway down. Remove catheter from package.
- With collection bag facing up, hold the catheter in one hand while pushing the bag away from the catheter with the other hand to break the paper band. Unfold collection bag. Leave the catheter on the lap, sink, or other flat surface.

Male

- Hold the penis with your non-dominant hand and don't let go until you have finished the procedure. Pull back foreskin (if present) and cleanse the glans and opening of urethra as instructed by your healthcare practitioner.

Female

- Spread the labia with two fingers of your non-dominant hand. Cleanse around opening of urethra as instructed by your healthcare practitioner.

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Male

- Remove the cap from the protective tip of catheter.
- Hold the catheter in one hand, and with the other hand advance the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.
- Grasp catheter below protective tip plate.

Female

- Part the labia with two fingers and insert the protective tip until the plate comes in contact with urethral opening.
- Relax the labia and use the fingers of your non-dominant hand to hold the protective tip in place as you insert the catheter with your dominant hand until the urine starts to flow.
- Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. Reposition the foreskin (if present).

Male

- Using your forefinger and thumb of your non-dominant hand squeeze penis gently to open the urethral opening. With your dominant hand, insert the protective tip until the plate comes in contact with urethral opening.
- With the penis stretched and in upright position, insert the catheter forward until urine starts to flow.
- Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. Reposition the foreskin (if present).

Female

- Part the labia with two fingers and insert the protective tip until the plate comes in contact with urethral opening.
- Relax the labia and use the fingers of your non-dominant hand to hold the protective tip in place as you insert the catheter with your dominant hand until the urine starts to flow.
- Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely.

To dispose

- Catheter and integrated collection bag may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET.
- Wash your hands.

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©2021 Hollister Incorporated. 913081-1121
Caution: Federal USA Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.