en No Touch Urethral Intermittent Catheter

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
en-es Cateter intermitente uretral sem toque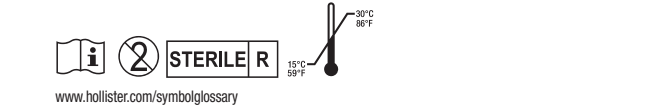






www.hollister.com/symbolgallery

USA
Rx Only
Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

CE 0050

Hollister®

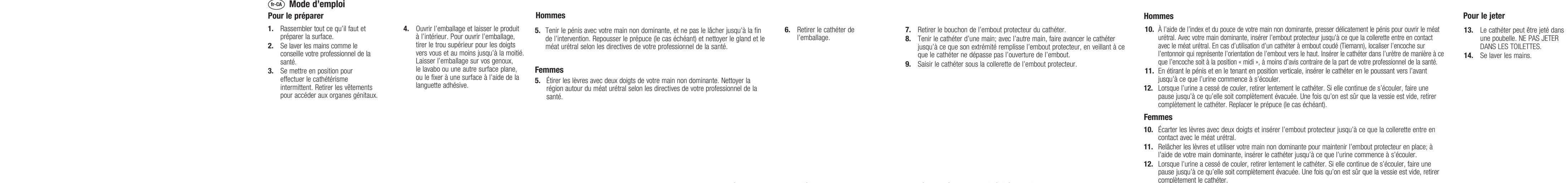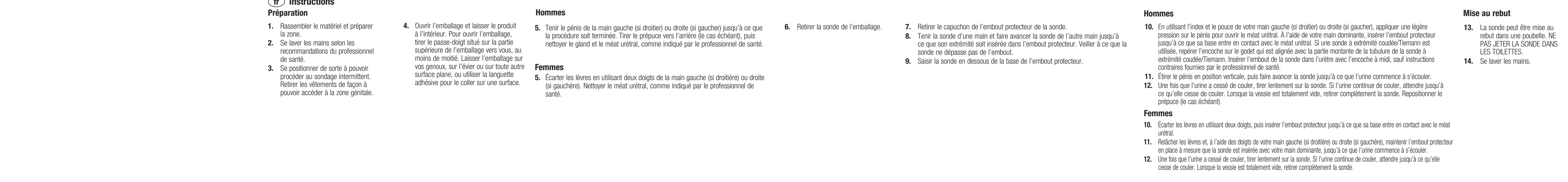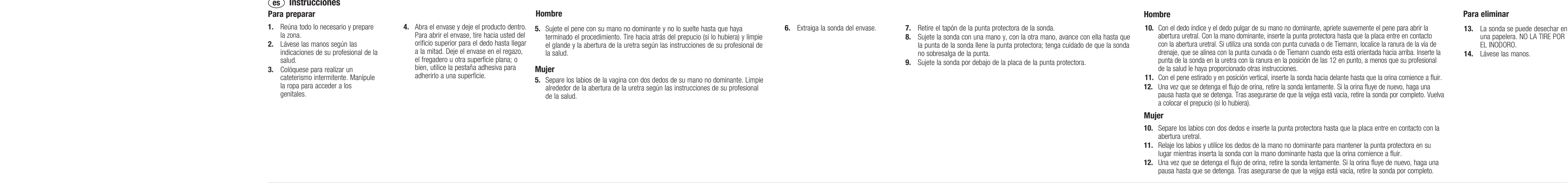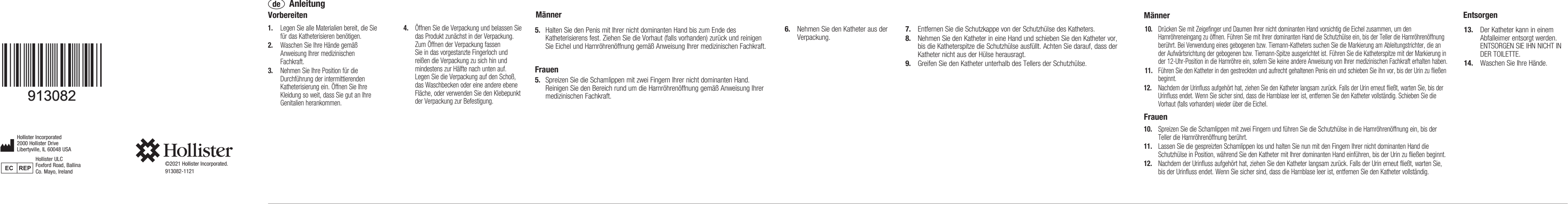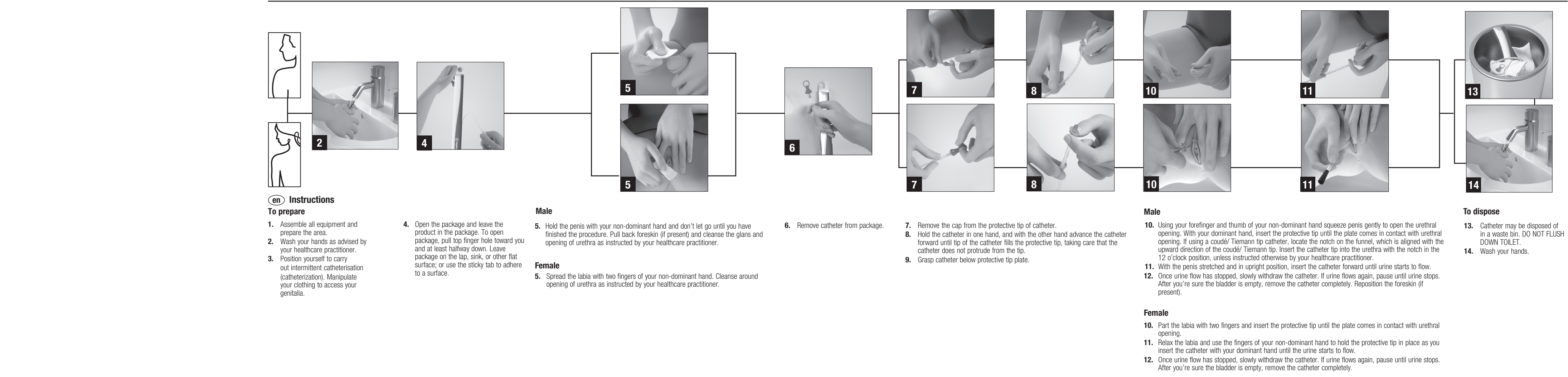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
Inhalt
Berührungsfreier Einmalharnröhrenkatheter zur intermittierenden Katheterisierung
Produktbeschreibung
• Ready-to-use hydrophilic intermittent catheter with colour (color)-coded funnel.
• Protective tip and sleeve supports no touch insertion technique.
• Adhesive tab for adhering packaging to a surface.

Catheter effective length: the length of the catheter that can be inserted into the urethra.			
20cm / 8" catheter	17 cm	170 mm	6.7"
40cm / 16" catheter	37 cm	370 mm	14.6"

Not made with natural rubber latex.
This product consists of: polyurethane sleeve, thermoplastic elastomer tip, and tubing not made with DEHP.
Indications and intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and paediatric (pediatric) patients who need to drain urine from the bladder.
WARNINGS
Do not use if package is damaged or opened. To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.
Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:
• Severed urethra
• Unexplained urethral bleeding
• Pronounced stricture
• False passage
• Urethritis – inflammation of the urethra
• Prostatitis – inflammation of the prostate gland
• Epididymitis – inflammation of the epididymis (testicle tube)
Self-catheterisation (catheterization) should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation (catheterization) frequency varies by person, the recommended frequency of your catheterisation (catheterization) should be provided by your healthcare practitioner. For any other questions about your catheterisation (catheterization), please contact your healthcare practitioner.
NOTE: Store boxes in a flat position and at normal room temperature. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

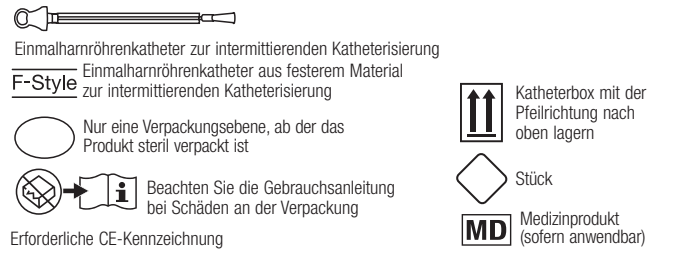USA
Rx Only
Caution: In the USA, this device is only for use by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

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