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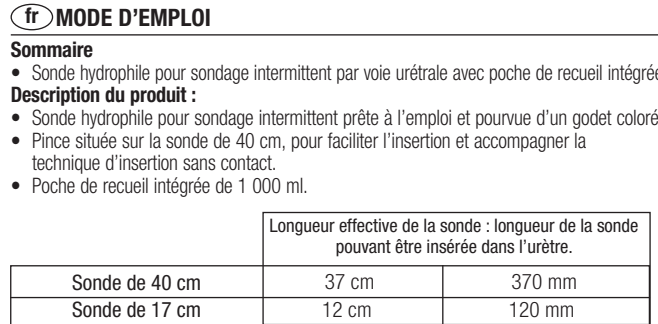
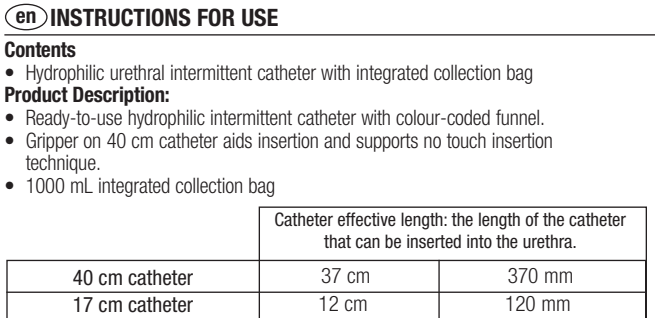
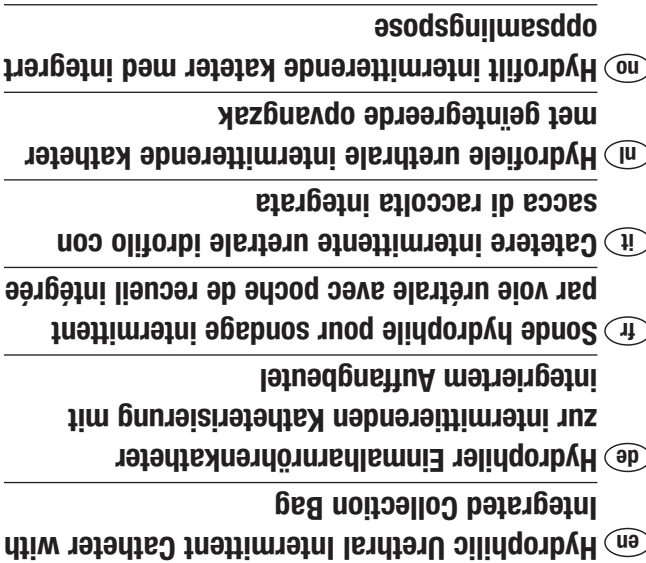
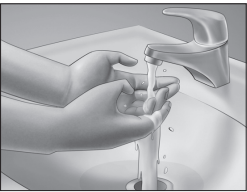
Snu for å se bruksanvisningen.

en Male Instructions

To prepare
Wash your hands as advised by your healthcare practitioner. Hold the penis with your non-dominant hand and don't let go until you have finished the procedure. Pull back foreskin (if present) and cleanse the glans and opening of urethra as instructed by your healthcare practitioner. Choose a position to catheterise that is most comfortable for you (standing, sitting on the toilet, in wheelchair).

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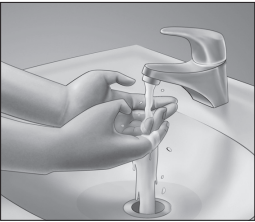
en Female Instructions

To prepare
Wash your hands as advised by your healthcare practitioner. Spread the labia with two fingers of your non-dominant hand. Cleanse around opening of urethra as instructed by your healthcare practitioner. Choose a position to catheterise that is most comfortable for you (standing, sitting on the toilet, in wheelchair or lying in bed).

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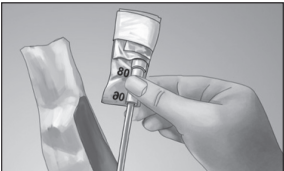
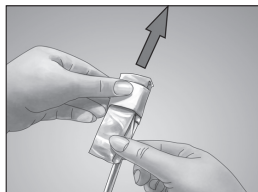
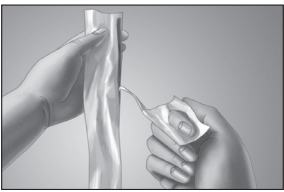
Open package, hold funnel and gripper or folded collection bag to remove catheter.

Release collection bag by slipping paper band away from the catheter and off the bag.

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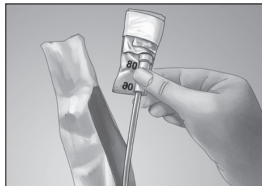
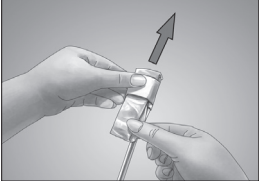
Open package, hold funnel or folded collection bag to remove catheter. Release collection bag by slipping paper band away from the catheter and off the bag.

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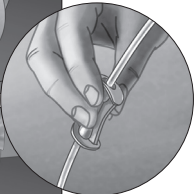
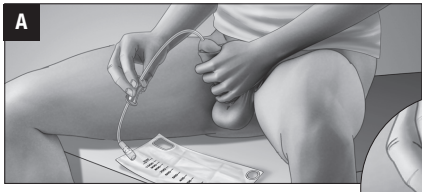
To use
Pull back foreskin if present.
A : Pull gripper away from the funnel. Hold gripper anywhere along the catheter, so you do not touch the catheter.
With the penis stretched and in upright position, hold gripper tightly to begin inserting catheter. Slightly release gripper, and slide it back towards funnel. Then hold gripper tightly again to advance the catheter. Repeat until urine starts to flow. Relax foreskin, if present.

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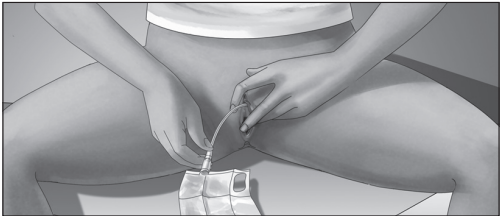
To use
Spread labia with two fingers while inserting catheter into the urethra. Hold funnel, without touching the catheter and insert catheter until urine starts to flow.

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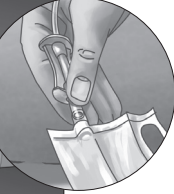
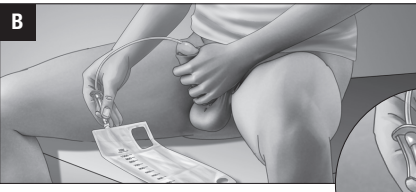
B : Alternative insertion option: keep the gripper 'docked' with the funnel. With penis stretched and in upright position, hold gripper/funnel and insert catheter into the urethra with one hand until urine starts to flow. Relax foreskin, if present.

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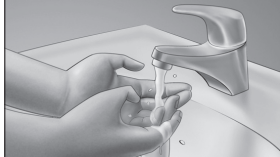
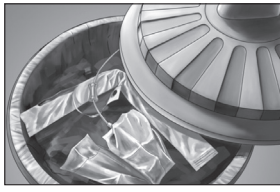
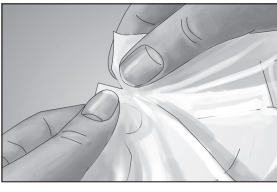
To remove and dispose (Please follow all local disposal regulations)
Once urine flow has stopped slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. After catheterising hold collection bag by handle. To empty the collection bag before disposing, tear at the arrow (Tear here to empty) to drain urine. Catheter and integrated collection bag may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. Wash hands.

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