en Instructions To prepare 1. Assemble all equipment and prepare the area. 2. Wash your hands as advised by your healthcare practitioner. 3. Position yourself to carry out intermittent catheterisation (catheterization). Manipulate your clothing to access your genitalia. 4. To open package, pull toward you at the notch. Remove the catheter from the package. Leave pack on the lap, sink or other flat surface. Lay catheter on opened package. 5. Spread the labia with two fingers of your non-dominant hand. Cleanse around opening of urethra as instructed by your healthcare practitioner. 6. Remove the cap from the protective tip of catheter. 7. Hold the catheter in one hand, and with the other hand advance the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip. 8. Grasp catheter below protective tip plate. 9. Part the labia with two fingers and insert the protective tip until the plate comes in contact with urethral opening. 10. Relax the labia and use the fingers of your non-dominant hand to hold the protective tip in place as you insert the catheter with your dominant hand until the urine starts to flow. 11. Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. 12. Catheter may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. 13. Wash your hands.												
---	--	--	--	--	--	--	--	--	--	--	--	--

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For reference only. Unauthorized use, disclosure or copying is prohibited.